ANPASSUNG

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Hersteller:

Snoring Relief Labs, USA. 342 S. Corona Street, Denver, CO 80209

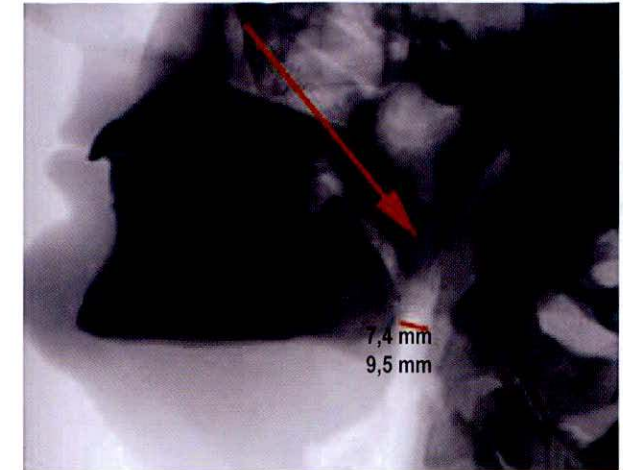
Registered, Adactive Marketing AB, 22763 Lund, Sweden, Class 1 Medical Device

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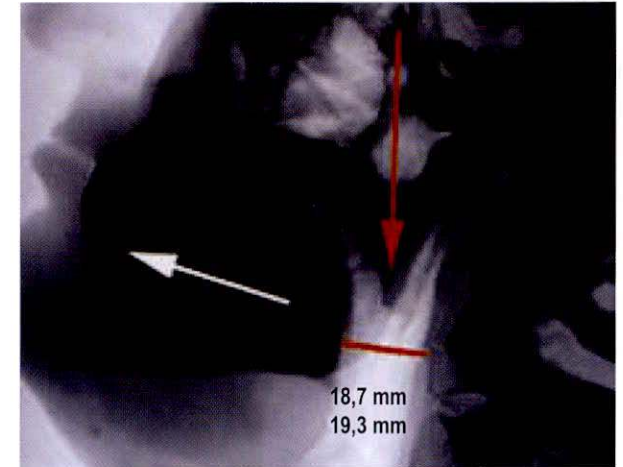
FITTING SNORBAN®

Preparation

You should practise the fitting procedure with cold water as described below before you begin the actual fitting. To correctly fit a SnorBan® you will need a clock with a second hand, a mirror and a glass of cold water.

Fitting

1. Boil sufficient water in a container deep enough to submerge the mouthpiece – except for the handle. **Do not leave the SnorBan® in the water whilst boiling.**

2. Remove the water from the heat source and, holding the handle, dip the SnorBan® mouthpiece in the near boiling water for **12-14 seconds**. This is all the time needed to soften the plastic. (NB! Exposing the mouthpiece to heat for more than 15 seconds will damage it).

3. Lift SnorBan® out of the water and shake off any excess water. The plastic hardens rapidly so you must move quickly. The plastic will feel warm but will not burn you. Using a mirror, open your mouth wide and place the SnorBan® in it, TOP side up (the upper half of SnorBan® has a small V-shaped indentation by the front teeth).

4. Place the SnorBan® over the bottom teeth, fitting the teeth cleanly between the channels. Push your lower jaw forward so that the teeth are edge to edge (see figure 2) and bite down firmly.

5. Next put your thumb and first finger in your mouth on either side of the device and pressing inwardly hard, push from the front teeth through to the back. At the same time push your tongue against the upper and lower inside front area of the SnorBan® mouthpiece. This will ensure a tight fit against your teeth.

Fig. 1
Normal tooth profile

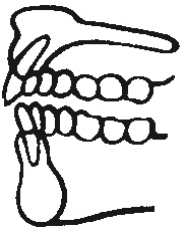
Fig. 2
Required tooth profile

6. When the mouthpiece has cooled, remove it from your mouth and dip it into a glass of cold water for about 10 seconds to set it to your custom shape.

7. Place the mouthpiece back into your mouth to evaluate the fit. Your lower jaw should be held in a comfortable forward position. If not, repeat the fitting, though this time the device should be heated for **only 12 seconds**. (You may remold the mouthpiece no more than 3 times).

8. Cut off the handle as close to the mouthpiece as you can. If a sharp edge remains, slightly heat a metal spoon and use it to roll (not drag) the sharp surface smooth.

9. If your snoring does not completely discontinue, the fitting can be repeated by extending the lower jaw slightly more forward.

Reference

For more information including a fitting instructions video and Frequently Asked Questions, please visit our website at www.snorban.eu.

Important

You must be able to clearly see a dental impression in the mouthpiece. If this is not the case, the mouthpiece must be reshaped. **DO NOT** bite on the rim of the CHANNELS, but **WITHIN** the channels. If you bite the rim, wearing the device will be painful, as this will tilt the jaw.

Here you can see a deep and clear dental impression. When the mouthpiece is correctly shaped, the pressure is distributed equally across all teeth and the mouthpiece will feel comfortable.

The importance of a good teeth indent within the SnorBan® mouthpiece is to provide support in holding the jaw forward.

Last revised 31 March 2017

Stop Snoring!

SnorBan®

Documented Effect
Soft and Comfortable Anti-Snoring Mouthpiece
Medical equipment for the treatment of snoring

SnorBan DK ApS, Dalgas Avenue 40B, 8000 Aarhus C,
Denmark, www.snorban.eu

USER INSTRUCTIONS

Package contents

Mouthpiece, storage box and user instructions.

Description

The SnorBan® mouthpiece is made of medical thermoplastic, which is softened when heated in warm water. In doing so, the SnorBan® mouthpiece is shaped to fit the individual user's set of teeth. SnorBan® is produced in the USA and the material is FDA approved and does not contain latex or BPA.

Application

SnorBan® can be used by adults for the treatment of snoring.

Operating principle

Snoring is caused by a narrowing of the airway in the throat (**X-ray 1**). When the airway is smaller, the velocity of air increases, while breathing causing the tissues to vibrate and make the sound we call snoring.

The patented SnorBan® mouthpiece holds your lower jaw slightly forward and opens the airway to the throat (**X-ray 2**). This position allows air to move through unobstructed, thereby eliminating vibration and any snoring noise!

Should not be used in the event of:

- Temporomandibular Joint Dysfunction (TMJ).
- Pronounced overbite or underbite.
- Loose teeth, unstable tooth crowns or advanced paradentosis.
- Full or partial dentures.
- Inadequate breathing through the nose.
- Severe epilepsy or chronic asthma.
- Children under the age of 18.

If any of the above conditions exist, you must consult your doctor before you use SnorBan®. If you are uncertain of the condition of your teeth, we recommend that you consult your dentist before using SnorBan®. In addition, we also recommend that you visit your dentist at regular intervals if you use SnorBan® every night.

Possible side effects

- Getting used to the mouthpiece takes 2 to 3 weeks. During this period you may feel a slight discomfort in the jaw area but this will disappear a few minutes after removing the SnorBan® mouthpiece.
- Increased saliva, which gradually normalises as you get used to the mouthpiece.
- In the first couple of weeks, SnorBan® may fall out of your mouth. This is normal and will not become a problem once you are used to using the mouthpiece.

In case of any persisting or severe side effects, stop using SnorBan® until you have consulted your doctor.

Problems breathing through the nose

If you catch a cold or suffer from allergies which lead to a blocked nose, you will not be able to use SnorBan® until you are able to breathe through your nose without problems. Tips: SnorBan® can be combined with a nasal dilator, Snooze, which eases nasal breathing or a nasal rinsing system, Nasaline, which reduces irritation in the nostrils and sinuses. Snooze and Nasaline can be purchased on the website www.snorban.eu.

Care

Daily: Add a couple of drops of Ecosym Gel on the SnorBan® mouthpiece and brush thoroughly with a soft toothbrush. Rinse the mouthpiece and leave to dry. Repeat the process after each use of your SnorBan®.

Weekly: Pour the Ecosym Forte cleaning fluid into the storage box that is included in your SnorBan® kit and add lukewarm water. Leave your SnorBan® in the cleaning fluid for 6-8 hours. Remove your SnorBan® from the cleaning fluid and rinse with water. The Ecosym Gel and Forte can be purchased on www.snorban.eu.

Warnings

- SnorBan® reduces snoring, which prevents sleep apnoea, but it is not a treatment against sleep apnoea. If you suspect that you suffer from sleep apnoea, you must contact your doctor.
- For a very small number of people, a condition called Temporomandibular Joint Dysfunction (TMJ) can be caused by using SnorBan®. If you have severe or persistent pain in the jaw joint, stop using SnorBan® and contact your doctor. Your doctor will then find out what may have caused this condition.

Distributor

SnorBan DK ApS, Dalgas Avenue 40B, DK-8000 Aarhus C
Telephone +45 50549804.
Email: support@snorban.dk

SnorBan® and accessories can be purchased in our web shop at www.snorban.eu.

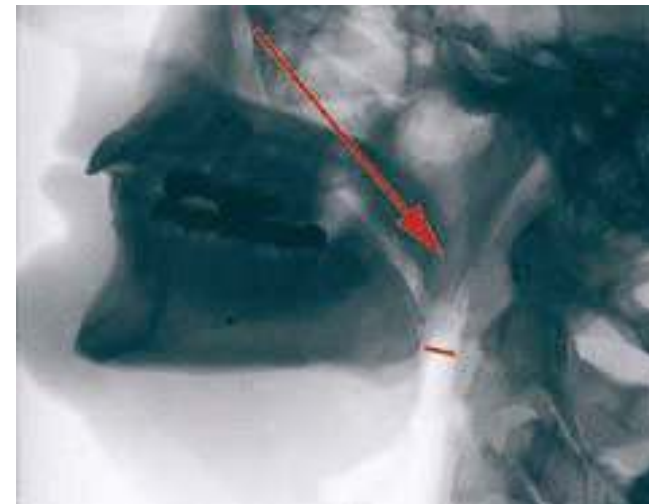
Manufacturer

Snoring Relief Labs, USA. 342 S. Corona Street, Denver, CO 80209.

Registered, SnorBan DK ApS, 8000 Aarhus C, Denmark, Class 1 Medical Device


PROBLEM

X-ray 1 – Without SnorBan®

The X-ray shows the airways. The space available for the passage of air is indicated by a red line.

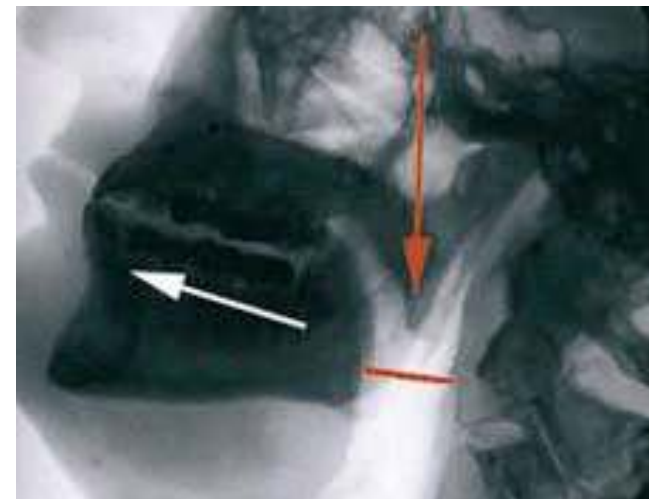SOLUTION

X-ray 2 – With SnorBan®

The X-ray shows the increased air passage when using SnorBan®, indicated by the red line.

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