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TEL (56)6229244

www.DRSAPPORO.com

USER MANUAL – DR SAPPORO PIANO ORTHOPEDIC PILLOW

Indications for use: supportive in the treatment of snoring.

Contraindications: It is advisable to consult a doctor before using PIANO pillow for severe sleep disorders, such as eg. sleep apnea.

How to use: The pillow should be placed under the cervical spine. Best to start from the back lying position (on the back), with head placed centrally, at the highest point of the pillow. Then - relax your body and allow the shape of the pillow to force the head to the lateral position (on the cheek). Note! Neck torsion effect is greatest if the head is initially positioned exactly at the highest point of the pillow. It is not recommended to lay head in the lateral parts of the pillow, as this may weaken the effect. You can sleep on a Piano pillow on the back with a twisted neck, or, in the side position with the face lying on the cheek. It is not recommended to sleep on a Piano pillow in the prone position (on the stomach).

Properties: PIANO Orthopedic Pillow supports the treatment of snoring. Specially designed shape of the pillow does not allow sleeping in a supine position, forcing the side position of the head. Thanks to this, during sleep the tongue does not fall to the rear and the air flow space increases, making it easier to breathe freely and reducing the symptoms of snoring. PIANO pillow does not eliminate the causes of snoring and only assists the treatment. PIANO pillow is made of a memory foam. The structure of this product has open cells that allow free flow of air.

Marking: PIANO pillow by DR SAPPORO is marked with international CE mark, confirming that, as a Class I medical device meet the health requirements laid down in Directive 93/42 / EEC

Cleaning: Regarding foam: if product needs to be refreshed, it is recommended to ventilate or wipe with a wet cloth after removing the pillowcase; washing in the machine may cause loss of properties; do not slap or swirl (this may lead to rupture); do not squeeze

Regarding pillowcase: washable at 40 ° C; Wash and dry inside out.

Storage: do not fold or break down, PIANO pillow must be stored and transported in a dry place.

Size: Universal 70x40x12 / 14cm / tolerance of +/- 5cm