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Foglietto illustrativo

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- habitual snoring, in particular tongue-based snoring
- mild to moderate obstructive (OSA), after medical diagnosis

Kontraindication:

- removable denture or poorly adhering partial denture
- temporomandibular joint pain
- gum loss or tooth loosening
- less than 10 teeth per jaw
- not applicable to persons under 18 years of age

If you are unsure about the stability of your teeth or hypersensitivity of your teeth (e.g. to heat), please consult your dentist before use.

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Warranty:

- the statutory warranty applies
- general wear is not warranted; service life of the snoring mouthpiece is approx. 18 - 24 months, in case of special stress (bruxism, oral acidity) it may be significantly shorter

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Adjustment of somnipax guard

1. Place the upper mouthpiece of Somnipax Guard (i.e. the mouthpiece with 2 hooks in the front) and the thermometer in a pot filled with approx. 8 cm of cold water and slowly increase the water temperature **a**.
- When the red thermometer fluid reaches the blue mark on the thermometer, the water is at the target temperature. Leave the upper part of the snoring mouthpiece over the upper teeth **b**.
- Align it so that the vertical line on the upper mouthpiece is in the

middle of the two front incisors. Now press the upper part of the snoring mouthpiece upwards against the upper jaw with both thumbs, displacing the pressure evenly on both sides of the jaw. Then take your hands out of your mouth and bite slowly but firmly. Next, using your fingers, press the inserted mouthpiece against your teeth. Hold the pressure for 60 seconds. Now rinse the mouth with cold water so that the mouthpiece can harden in the mouth. The upper mouthpiece is now ready for use.

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Effets secondaires possibles:

- Sensitivity of teeth in the morning after waking up which usually disappears within a few minutes (chattering the teeth can also help get rid of the sensitivity)
- Pain at the base of the ear - usually disappears a few days after first application
- excessive salivation - usually disappears a few days after first application
- Change in the contact points of the upper and lower jaw (dental occlusion) - as a rule, the lower jaw quickly returns to its natural position in the morning; if symptoms persist, a doctor should be consulted
- Now repeat the procedure to adjust the lower mouthpiece (i.e. the mouthpiece with the 2 hooks at the back). However, please insert the upper part that has already been fitted before beginning the fitting of the lower mouthpiece. Remove the lower mouthpiece from the hot water with a spoon, shake to completely remove the excess hot water. Also check that the mouthpiece does not burn your mouth, by testing the temperature against your wrist, before in-

If in doubt, seek medical advice.

Note on adjustment:

- the better the quality of the dental impression, the better the fit of the mouthpiece
- the adaptation can be repeated several times if necessary, as the material is thermoplastic and can be heated several times without losing its properties

• Any rough edges caused by adjustment can be carefully removed with a sharp knife

Note on the correct belt length / protrusion:

- The patient should use strap number „+6“ for the first nights after the adjustment; then, if necessary, use shorter/longer straps. The number on the strap length stands for the average mandibular adjustment (in mm) that can be achieved with the strap. The shorter the strap, the greater the mandibular adjustment
- If strap 6 is too tight (i.e. you experience an unpleasant pulling sensation in the jaw), a longer strap should be used, i.e. a strap with a smaller number (e.g. „4,5“)
- If snoring / OSA still occurs when strap 6 is used, you should use a shorter strap, i.e. increase the mandibular adjustment (e.g. „7,5“)
- 4,5 - 7,5 mm of mandibular adjustment is the bandwidth of the most frequently used strap lengths

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