**Gebrauchsinformation/Manual/Notice d'utilisation/
Indicazioni d'uso/Instrucciones para el uso/
Handleiding/Käyttöohje**

DEUTSCH 5

ENGLISH 6

FRANÇAIS 8

ITALIANO 10

ESPAÑOL 11

NEDERLANDS 13

SUOMI 15

DEUTSCH 18

ENGLISH 20

FRANÇAIS 22

ITALIANO 23


ESPAÑOL 25

NEDERLANDS 27

SUOMI 29

Lieferumfang/Delivery includes/Contenu de la livraison/Ambito di consegna/Alcance de la entrega/Omvang van de levering/Pakkauksen sisältö

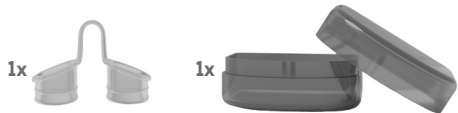- DE:** 1 Nasenspreizer gegen Schnarchen, 1 Aufbewahrungsbox
EN: 1 Nasal dilator for snoring, 1 Storage box
FR: 1 Dilatateur nasal anti-ronflement, 1 Boîte de rangement
IT: 1 Dilatatore nasale contro il russare, 1 Custodia
ES: 1 Dilatador nasal antironquidos, 1 Estuche
NL: 1 Neusspreider tegen snurken, 1 Bewaardoosje
FI: 1 Sieraintenlaajentaja kuorsauksen ehkäisyyn, 1 Säilytyskotelo

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Manual • ENGLISH

The somnipax breathe nasal dilator facilitates nasal breathing by gently dilating the nostrils. Improved nasal breathing prevents snoring and ensures a more restful sleep.

The somnipax breathe nasal dilator is safe to use, unlike some nasal sprays or drops there

are no side effects or habituation effects.

somnipax breathe is made of soft, elastic medical grade plastic. It is transparent and inconspicuous. All rough edges of the nasal dilator are rounded so that the sensitive mucous membranes are not irritated.

The best way to use somnipax breathe is as follows:

- 1.** First clear your nasal passages, to do this you can use a nasal douche (such as NoseBuddy).
- 2.** When using somnipax breathe for the first time, you should stand in front of a mirror to insert them. This will make the insertion easier.
- 3.** Remove the somnipax breathe from the packaging. Hold the somnipax breathe by the

u-shaped connector with the cylindrical components facing upwards.

4. Now gently insert the cylindrical components of the somnipax breathe into your nose by applying light pressure until the u-shaped connector rests on the nasal columella. If you have a very small or narrow nose, you may not be able to insert the somnipax breathe too deeply.

5. The somnipax breathe should not cause you any pain once inserted. Nevertheless, you will initially perceive it as a foreign body in your nostrils, this sensation will diminish steadily after a short accustomation period.

Cleaning instructions:

The somnipax breathe can be used up to 60 times. Simply wash it thoroughly with warm water after each use and then dry it with a

clean towel.

When cleaning, do not use abrasive or corrosive detergents, as these can damage the material of the nasal dilators and should also not come into contact with the sensitive mucous membranes in the nose. You should store the somnipax breathe in a clean, dry place.

Important information:

Do not use somnipax breathe if you have issues with insertion due to an asymmetrical nose or you are known to suffer with blocked or obstructed nasal breathing. In addition, the nasal dilator is not suitable for the treatment of obstructive sleep apnoea. In these instances, please seek advice from your ENT doctor or a sleep specialist before use.

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Manual • ENGLISH

The somnipax pure is a discreet nasal dilator with an air filter. It is inserted into the nostrils to improve nasal breathing, cleanse the air you breathe and reduce snoring. The replaceable filters of the somnipax pure filter out particles from the air we breathe in, such as house dust or pollen. The somnipax pure thus prevents the inhalation of dirt, pollen or other allergenic particles from the air, which can irritate the mucous membranes and cause them to swell. The somnipax pure is made from pleasantly soft, medical grade plastic. Since the nasal dilator acts mechanically, it does not cause any side effects even with regular use.

This is the best way to use the somnipax pure is to:

- 1.** Clear your nasal passages first, a nasal douche (like the NoseBuddy) is suitable for this purpose.
- 2.** When using somnipax pure for the first time you should stand in front of a mirror. This will make the insertion easier.
- 3.** Remove the somnipax pure and two filters from the packaging. Now insert the filters into the nasal dilator. To do so, simply push the filters into the channels inside the cylindrical shaped dilators. Hold the somnipax pure by the connecting loop so that the cylindrical components point upwards.
- 4.** Now gently insert the somnipax pure into your nose, using light pressure, until the connecting loop rests on the columella of the nose

(the area between the two nostrils) without pressure. If you have a very small or narrow nose, you may not be able to insert the somnipax pure quite as deeply.

- 5.** The somnipax pure should not cause you any pain when worn. Of course, initially you may experience a sensation of having a foreign body in your nose at the, but this will disappear after a short period of acclimatisation.

Cleaning instructions:

The somnipax pure can be used up to 60 times, but you should replace the filters after three usages at the most. Please remove the filters immediately, when you remove the somnipax pure from your nose after use. Then wash the somnipax pure with lukewarm water and dry it with a clean towel. You can also clean the filters twice by rinsing them with lukewarm water

and letting them dry afterwards. After the third use of the filters please dispose of, or replace them. Do not use any harsh or alkaline cleaning agents when cleaning, as these corrode the material and they should also not come into contact with the sensitive nasal mucous membranes. Store the somnipax pure in a clean and dry place when not in use.

Important information:

Do not use somnipax pure if you have issues with insertion due to an asymmetrical nose or you are known to suffer with blocked or obstructed nasal breathing. In addition, the nasal dilator is not suitable for the treatment of obstructive sleep apnoea. In these instances, please seek advice from your ENT doctor or a sleep specialist before use.

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